



Medicare Sterile Saline Solution

Gentle Wound & Eye Wash Spray



FACT BOX

Product Code:
MD1221

Product Category: Wound and skin care, First Aid, OTC.

Composition: Sterile 0.9% NaCl solution, Sterile 0,9% NaCl phosphate buffered solution or alternatively sterile Ringer solution

Product Approvals:
Registered according to MDD (Medical Device Directive, 93/42/EEC, Class IIa)

Shelf Life:
3 years from production date

Packaging:
Standard can size is 100ml.

Precautions: Avoid direct contact with the surfaces of the wound/eye.

Medicare Sterile Saline Solution is perfect for rinsing and cleansing away dust and dirt from wounds and eyes.

The spray physically removes foreign bodies, blood, reduces pathogens and dead tissue whilst also moistening the tissue in the process which contributes to wound healing.

Bag-on-valve technology allows you to use every last drop with a longer shelf life. No pumping motion needed.

FEATURES

- Removes dust and dirt from wounds and eyes.
- Hypoallergenic.
- Sterile until the last drop.
- Can be used with the canister in any position.

DIRECTIONS:

Spray a small quantity through the nozzle before use. Hold the can in any angle, at a distance of at least 10cm from the wound or eye to be treated. Spray across the open wound/eye to thoroughly cleanse and irrigate.

RECOMMENDED USE:

A non-medicated sterile saline solution for rinsing and moisturising eyes and wounds.