



LIFESENSE®

# Medicare Lifesense A1 Blood Pressure Monitor

Accurate, lightweight & one-touch quick measurement



## FACT BOX

**Product Code:**  
MD1814

**Product Category:**  
Diagnostics.

**Product Approvals:**  
CE marked.

**Memory:** 2 x 60 memory groups per user.

**Cuff Size:**  
22cm- 32cm.

**Power:**  
4 x AAA Batteries.

Medicare Lifesense® A1 Blood Pressure Monitor is an easy to read, automatic upper arm blood pressure monitor designed with both comfort and accuracy in mind. The slim, portable design makes this a great product to bring with you when travelling or to discreetly store in the home. The high quality blood pressure measures during inflation, making the measurement time shorter and a lot more comfortable.

## FEATURES

- Digital LCD display (60mm x 41mm).
- Large soft touch buttons.
- Large text for high visibility.
- Measure-during-inflation technology.

## DIRECTIONS:

1. Make sure you are relaxed before you begin.
2. Roll up your sleeve and fasten the cuff 2-3cm above your elbow bend. Rest your arm with your palm facing up. Try to avoid moving or talking during measurement.
3. Press the 'Start/Stop' button to begin measurement. It will automatically stop when the measurement is completed.
4. Your results will appear once the device has stopped.

## RECOMMENDED USE:

Blood pressure monitoring.

**Barcode:** 5099390417203



@Medicarefirstaiduk



@Medicarefirstaid



@Medicare1staid