



# Medicare Non-Contact Infrared Thermometer

Point. Scan. Read



## FACT BOX

**Product Code:**  
MD1831

**Product Category:**  
Diagnostics.

**Product Approvals:**  
CE marked.

**Power:**  
x2 AAA batteries.

**Precautions:**  
For accurate results, allow the thermometer to acclimatise to the room temperature for 15-20 minutes before using it. It is important to allow 3-5 seconds interval between each measurement.

 **Fast, 1 Sec Reading**

Medicare Non-Contact Infrared Thermometer is a fast, accurate infrared thermometer with convenient features to help you monitor temperature with ease. It has 3-in-1 measurement options for body, surface and room temperature.

## FEATURES


- Fast, 1 second measurement.
- Measures without skin contact (3-5cm).
- Fever alarm.
- °C/°F switchable.
- 32 memory sets.

## DIRECTIONS:

Hold the thermometer 3-5cm away from forehead. Aim to take temperature at middle of forehead for accurate results.

## RECOMMENDED USE:

For temperature monitoring. The colour coded display will help to understand the temperature reading.

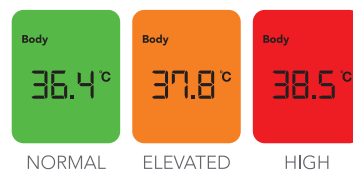

**Barcode:** 5099390414103