



Medicare Wart & Verruca Cryotherapy Pen

Freezes warts and verrucas to the core



FACT BOX

Product Code:
MD1915

Product Category:
Footcare, OTC.

Composition:
Dimethyl Ether (DME)

Product Approvals:
Registered according to MDD
(Medical Device Directive,
93/42/EEC, Class IIa).

Shelf Life:
2 years from production
date, do not use after
3 months of first use.

Packaging: 38ml.

Precautions: Extremely
flammable, pressurised
dispenser. Keep away from
heat, hot surfaces, sparks,
open flames and other ignition
sources. For external use only.
Avoid contact with eyes.

Medicare Wart & Verruca Cryotherapy Pen is a cryosurgical product for the treatment of warts and verrucas on the hands and feet. The pen immediately freezes the wart during treatment. A single application of the cryotherapy pen is sufficient to treat the concerned area.

FEATURES

- Begins to work from the first use
- Easy to use application pen
- Visible results within 2 weeks

DIRECTIONS:

Place the round opening of the applicator cap directly over the affected area and hold the dispenser at a 30° angle for the treatment to be most effective. Press the bottom of the applicator firmly with the thumb to dispense once. Repeat this up to 5 times depending on the size of the wart/verruca. Hold the applicator in place for 10 seconds before removing it from the skin.

RECOMMENDED USE:

Use on warts and verrucas on hands and feet.
Not suitable for children under the age of 4 years.

Barcode: 5099390416961



@Medicarefirstaid



@Medicarefirstaid



@Medicare1staid