



Medicare Adjustable Adult Arm Sling

Assists in healing injured arms, shoulders & elbows



FACT BOX

Product Code:
MD521

Product Category:
Home Health.

Composition:
85% Polyester, 15% Nylon.

Product Approvals:
CE marked.

Size:
One size fits all.
Fits right or left.

Precautions: If the support is being used following a sprain, seek medical advice if no improvement in the condition is seen within 72 hours. If the sprain is severe, or if in any doubt, seek medical advice immediately.

Medicare Adjustable Adult Arm Sling provides general arm support and restricts movement to assist in healing sore or injured arms and elbows. Allows injured shoulders or arm muscles to relax. It is also a useful aid following shoulder surgery or a shoulder injury. The arm sling is exceptionally easy to use and very simple to adjust, thanks to the adjustable shoulder strap.

FEATURES

- Suitable for men and women.
- Adjustable padded shoulder strap.
- Custom and comfortable fit.
- Helps relieve pain and prevent re-injury.

DIRECTIONS:

1. Loosen shoulder strap and place elbow in corner of sling.
2. Slide shoulder strap over head and rest on opposite shoulder. Position shoulder pad on top of shoulder.
3. Fasten and adjust strap so arm rests in a comfortable, horizontal position.
4. To remove, simply slide shoulder strap over head.

RECOMMENDED USE:

Hand wash in cool water with mild detergent. Rinse thoroughly and air dry. Do not use bleach or fabric softener.

Barcode: 5099390407167